

OCTAGON+
Face & Body



TIMELESS BEAUTY



A complete body and beauty studio in one device

OCTAGON+
Face & Body



**Stimulates
hair regrowth**

**Reduces and
fills wrinkles**

**Gives fullness to
lips and cheeks**

**Chinese
facelift**

**Reduces couperose
and pigmentation**

Slim/Detox

**Minimises
stretch marks**

**Minimises scars
and pain in scars**

**Lymphatic drainage/
cellulite**

Reflexology

INTRODUCTION7

WE OFFER FOUR TYPES OF THERAPIES IN OCTAGON+:

- 1. *BEAUTY THERAPY* 8
- 2. *BODY THERAPY* 10
- 3. *SCAR AND PAIN THERAPY* 12
- 4. *HAIR LOSS THERAPY (OPTIONAL)* 13

TESTIMONIES FROM SATISFIED CUSTOMERS AND THERAPISTS..... 14

CERTIFICATION COURSES 16

OCTAGON+ ACCESSORIES 18





OCTAGON+

Face & Body

With all the technologies available in *Octagon+* at your fingertips, more than 8 different technologies, you can provide your clients the best of care. By augmenting your skills and bringing them to a higher level, you stand out from your competitors. Above all this professional machine and system is time tested and researched, applying modern technology and embracing Traditional Medicine wisdom, gives you results you and your clients will be amazed by. Don't just take our words for it. Read on our Customer Panel for their stories.

The educational courses are intensive and rewarding. You will become an expert in skin therapy, body therapy, scars and scar-related pains. The courses are divided into 4 modules. The 4th module for hair loss is optional and the equipment for hair loss treatment are optional purchases.

Our classical Chinese Facelift is the basis of all our facelift treatments. The face and neck is rolled and worked on with two brass rollers or pins passing weak electrical impulses designed to increase blood flow, energising the cells and stimulating production of collagen. It is enhanced by deep tissue massage with the silver threaded gloves. The experience is relaxing and nothing is more satisfying than to know that your client is happy to see the results in the mirror afterwards. Happy clients become loyal clients and they are the backbone of your business.

If you are in the body therapy business, our Slim Detox treatment is a special treat for you and your client. It detoxifies, slims and reduces cellulite. A weight loss of up to 1,4kg has been recorded for just a 40-minute treatment session. What is impor-

tant is that the client feels more energised and that the body's metabolism is stimulated.

We have been in the forefront with scar treatment. Our method relieves pain from scars and help with the healing quickly. Psychological barriers from the scars are broken down. Many women develop stretchmarks from pregnancy, and these are reduced by 50% with 10 treatments.

To top it *Octagon+* offers hair enthusiasts treatments to stop hair loss, boost hair regrowth by using a combination of our specially designed current, LED lights and ultrasound scrubber together with our specially developed hair products.

Octagon+ Face & Body is an interesting machine as you can mix and match the different technologies to give the best results to your clients. In addition Electro-comb and Cryotonic are optional devices that are add-on options.

» Happy clients become loyal clients, and they are the backbone of your business

Octagon+ therapies

Chinese facelift™

Our face is our luck, say the Chinese

Combined with Dui's herbal blend, a synergistic effect is achieved. By improving energy circulation, we can achieve a natural facelift without the use of surgery or syringes.

By working on acupuncture points on the face, we can achieve a facelift, reduce wrinkles and at the same time strengthen the digestive system. The muscles become tighter and the skin stronger, thicker and more youthful.



areas and add new energy. This will cause the bags to be much smaller.

Wrinkles/Anti-Aging

The goal of the treatment is to reduce the tension in the skin, increase blood circulation and strengthen the connective tissue. The purpose is to delete wrinkles and penetrate ingredients into the skin to fill in wrinkles. It will also lead for the skin to rejuvenate.

Turkey Neck


The treatment tightens and shapes muscles and skin, and smoothes out wrinkles in such a way that the client's skin looks smoother and appears younger.

Eyebags

Poor circulation in the kidney area and pituitary gland will result in bags under the eyes. This treatment will reduce swelling in these

Muscle Toning

This is done by the device sending weak impulses to the muscles via brass rollers, silver wire gloves, electro pads and/or yellow LED light. The wavelength stimulates new production of collagen, elastin and hyaluronic acid.

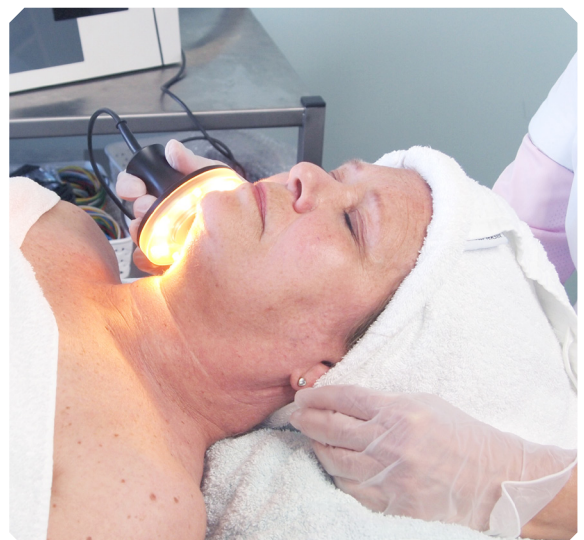

Octagon+ therapies



Treatment of acne

The treatment has a deep cleansing and healing effect on the skin. It kills the bacteria that are prevalent in the pustules and thus stops further infection.

Sun-damaged skin

The skin becomes thin and inflexible with pigment spots and couperose:

Pigmentation

Octagon+ pigment treatment reduces over-pigmentation in the skin from free radicals. It increases blood circulation, improves the blood's own cleansing process and helps the skin to release the excess pigment. This leads to a healthy, pretty, balanced and fresh skin.

Couperose

This treatment cleanses the skin tissue in depth and removes stagnation in the capillaries which gives the characteristic red color. It also stimulates cell renewal.



Octagon+ therapies



Skin Rejuvenation

Treatment with gloves that contain silver threads tones and shapes muscles and skin on the body and face, as well as smoothing out wrinkles. The silver threads in the gloves increase the effect of the treatment by increasing cell renewal and the repair of the skin.

Cellulite


A condition caused by poor blood circulation and imbalance of salt in the body. Octagon+ and Dui's herbal blend increases circulation and remineralizes through the skin.

Fat and electrolipolysis


Electrolipolysis treatment uses special frequencies (low frequency). The electrical stimulation takes place in the middle of the area with lipodystrophy using self-adhesive electrodes that are attached to the skin.

Lymphatic drainage

Performed using weak current and systematic stimulation of lymphatic vessels and fluids. With only 50 minutes of treatment, noticeable results can be achieved in a very effective way.

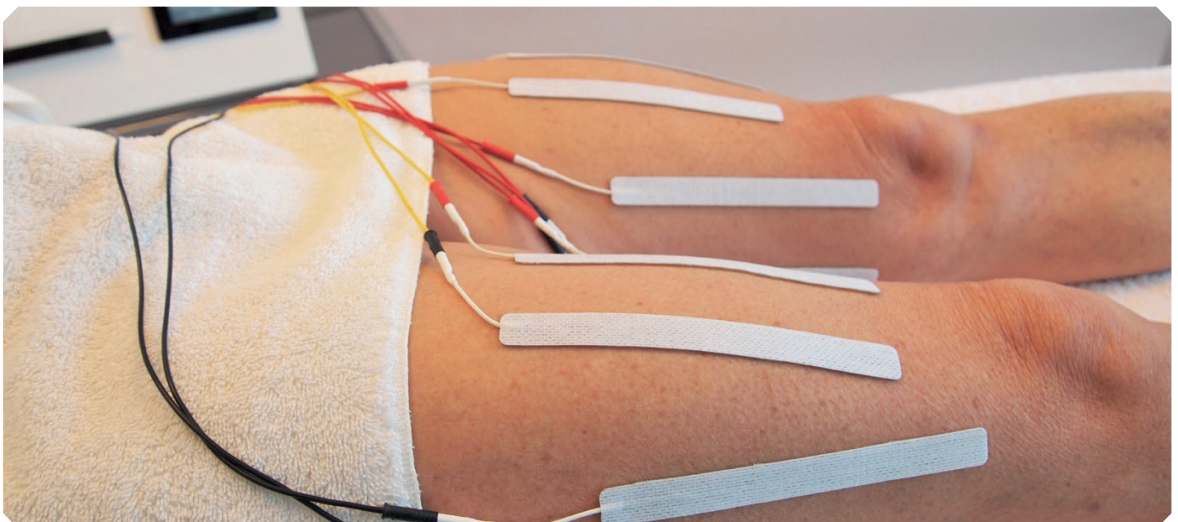

Octagon+ therapies

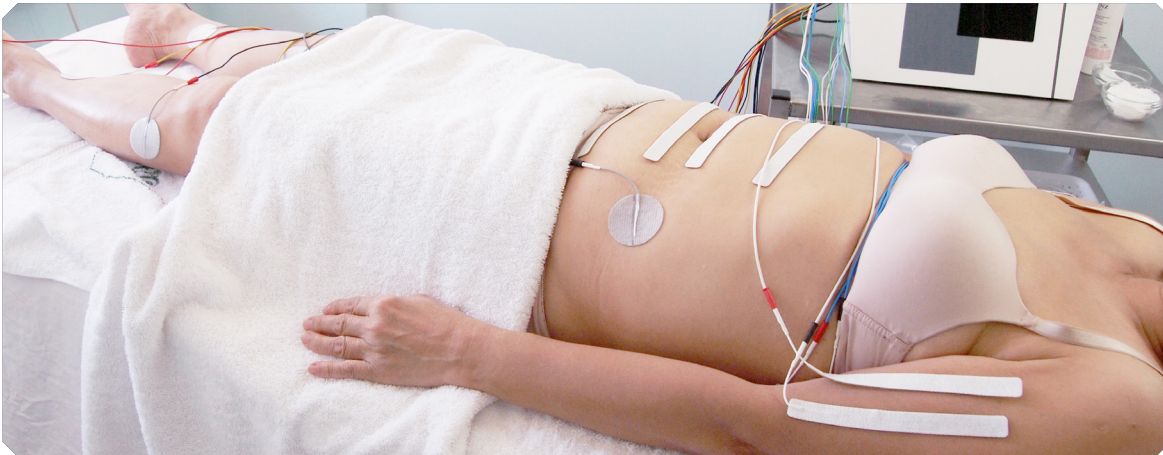

Slim/Detox

Slim/Detox is electrotherapy that uses acupuncture points to stimulate and balance organs for the purpose of detoxifying the body and achieving weight reduction. At the same time the lymphatic system, which is responsible for immunity, is supported. Weak electrical impulses sent via the nervous system's signals to the central nervous system include the brain, which in turn will send signals to the respective organs. In this feedback system, the internal organs will be stimulated. The organs supported are kidneys, spleen, stomach, intestines and liver.

At the same time, electrodes are placed on areas for local slimming effect, for example on the abdomen, hips, arms, thighs and back. This process is called electrolipolysis.


The total effect of this treatment is clear from the beginning. Both weight loss and size loss are measurable after a 50 minute treatment.

Sculpting and muscle stimulation

This form of electrotherapy signals to the nerves in the muscle attachments that the muscles should contract. It stimulates the metabolism in the muscles, cleanses metabolite waste, prevents the storage of fat and helps to tighten the skin. By increasing the metabolism in the muscles, glucose is broken down into energy molecules (ATP). This will lead to combustion of calories.


There are several uses of muscle stimulation: muscle toning, dieting and pain treatment on the body and face. A short session of 40-50 minutes will reduce the circumference and increase the metabolism. The body will retain and increase the results after repeated treatments.

EFFECT OF CURRENT ON FAT AND CELLULITE

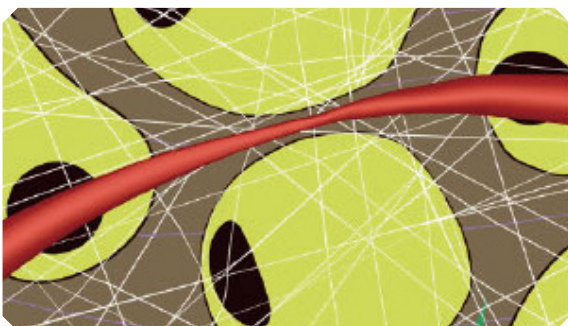

Before treatment

The fat cells have increased so much in volume that they squeeze the blood vessels and prevent normal blood circulation.

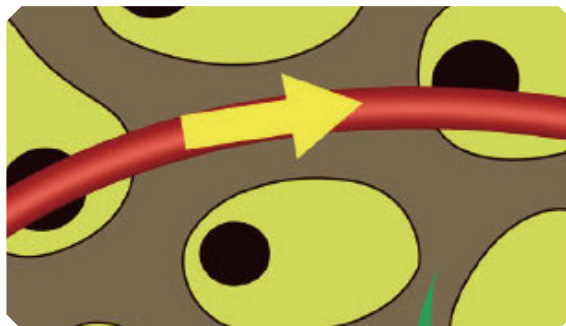

After treatment

The treatment has reduced the volume of fat cells so that normal blood circulation is restored. This leads to a reduction or elimination of cellulite.

Octagon+ therapies



Acne scars

Treatment with Dui Clinique current aims to kill bacteria in pustules (pimples), stop inflammatory conditions and initiate the healing process. The treatment also lightens scars after acne that have become black, brown, red or dark.

Surgical scars


This treatment accelerates the healing process of scars from operations, accidents and burns etc. The treatment makes scars and keloids (benign skin tumor, most often in scars) less visible, removes itching, burning and pain in and around the scar.

Stretch marks

Treatments with Octagon+ can regenerate the skin so that no one has to live with these

scars anymore. The stretch marks become smaller and tighter.

Octagon+ uses a low electrical energy to increase the body's regenerative function. This can increase the production of new nerves and the production of new cells in both the epidermis and the dermis. Circulation in the area is increased and waste products are removed. This stimulates healing.

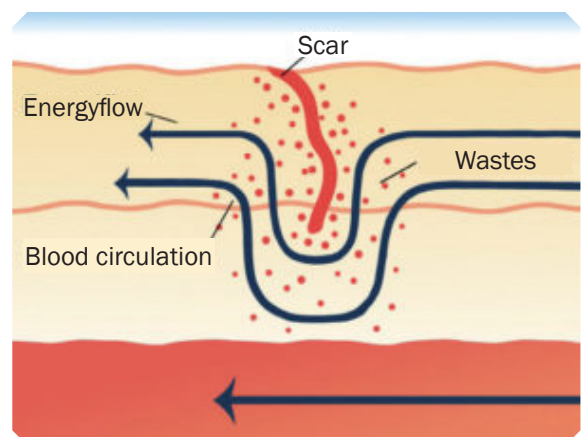

Octagon+ therapies



The most common type of hair loss in men and women is androgenic alopecia. About half of the population has significant hair loss. This is due to genetics, nutrition and stress. The underlying cause is inflammation.

Octagon+ has various solutions to reduce inflammation, improve lymphatic flow, improve circulation in the scalp and energy production at the cellular level. The low electrical stimulation in Octagon+ via rollers, silver-threaded gloves and electric comb with LED lights, is in accordance with the Arnd-Shultz law, which states that low electrical impulses stimulate physiological activities.

The choices between ultrasonic scrubber, rollers, electro comb and cryotonic probe give therapists the opportunity to provide varied treatments to their clients.

Androgen alopecia in men and women

This is largely due to genetics, and in general hair loss in humans is usually due to this. It can start as early as the late teens and early 20s. The earlier the treatment process starts, the greater the chance for clients to keep their hair.

Alopecia areata


This is caused by an overactive immune system. About 3% of the population has this type of hair loss. Working with both electrical impulses and cold will improve lymphatic power and eliminate metabolic wastes.

Traction alopecia

Hair loss due to the hair being pulled back too tightly, which damages the hair follicles. This can lead to involuntary hair loss later in life, especially around the hairline and at the temples. This is reversible and can be processed with power and LED light.

Scalp Conditions

With the combination of our products and the ultrasonic scraper, we can normalise dry, oily, dandruff and seborrheic dermatitis.

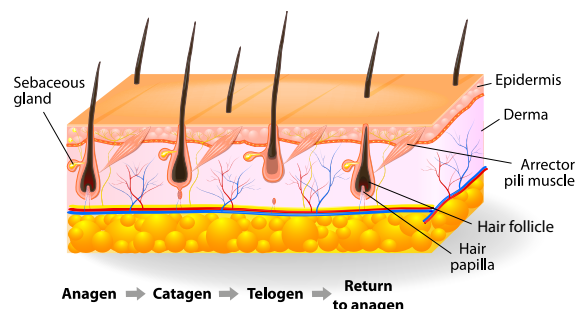



Happy man back in full time job

Vigdis Andrå, Dui Clinique & Frisør - Elverum

- A customer got a screwdriver into his hand, whereupon the ring finger and tendon were destroyed. After nine treatments with Dui Clinique current and a cover with cream mixture from Dui, the finger is straightened and the scar is softer. Now he can place his hand on the table with as straight a finger as the other fingers on both hands. A very happy man which no longer is inhibited by his hand and finger, pain is gone and he is in full time job again.

A feeling of relief, increased energy and joy

Tone Brøyn Wilhelmsen, Skap Rom - Hokksund

- As a Cryo 21 therapist, I like to combine Octagon and Cryo 21 to get results a little faster. I also have some exciting experiences with Octagon treatments and scars.

It is a known fact that scar tissue in the skin, after an operation, for instance, easily creates a brake in the fat freezing process. This is because the skin often builds up an extra layer of fat cells in the area around the scar, and develops a kind of "hanging shackle" in connection with the scar. This is what happens when the body's own biological currents are clipped over. The combination of light current treatments with the Octagon rolls and fat freezing treatments with Cryo 21 are clearly the best way to solve this type of challenge. As a positive bonus effect of the scar treatments, the tensions in the scar are simultaneously released and the customer can suddenly get a feeling of relief, increased energy and joy afterwards! A good thing to make the customer aware of!



Does not recognize the stomach

Hilde Gunns Velvære/Strømmen velvære - Arendal

- Three Octagon treatments and the stomach is completely changed. She had very hard fat and got Scar and Cellulite treatment and some LED light and ultrasound at the end, now she does not recognize her stomach! She is very happy!



Octagon has many functions for face and body

Kari Ellingsen, Cryoklinikken Hønefoss

- My background has previously been as an optician for approximately 20 years and with office and administrative tasks for several companies.

I coincidentally came to read about Cryo 21 when I wanted to start my own business, and found this very interesting - and with a serious approval of the device.

I have been working with Cryo 21 since May 2016. After a while, I wanted to be able to use additional treatments to be able to get better results. Octagon has many functions for face and body which can be combined with Cryo 21. Octagon has functions such as Led/Ultrasound and current. Programs for Slim/Detox and Lymphatic Drainage which are important for managing fluid retention and strengthening the body's internal organs. I also use Octagon to work with scars.

The biggest challenge in establishing this business is to learn and get acquainted with the programs.

Can recommend this machine to those who want to put join the programs and learn more about the body and health.



Scar treatment with Octagon

After only one treatment I feel so much better

Customer at Hilde Gunns Velvære/Strømmen velvære - Arendal

- Hi! Today I had my first treatment with Octagon-rolls. Have gone with pain in my scar after an operation, so this I must tell all of you who are struggling: come and try! After only one treatment I feel so much better, so after a few more treatments I will probably be completely fine. So THANK YOU lovely HILDE, I am so grateful for the help! Hug.



Octagon gives amazing results

Rosalyn Jensen, Rosalyn Helse & Velvære kropp og hudpleie - Stavanger

- We have tried out Slim/Detox on seven existing customers and all have lost several kilos. This is the result of a customer of 65 years who has taken ten Arctik 21 and four Octagon Slim/Detox treatments and lost a full 27 kg!

Certifications

1. Beauty therapy
2. Body therapy
3. Scars and pain therapy
4. Hair loss therapy (optional)

1

Beauty therapy

- Skin anatomy
- Skin analysis
- Paramedical skin analysis
- Face zones according to TCM
- Skin, hormones and health
- Client consultation
- Chinese facelift™
- Turkey neck
- Anti-aging
- Skin Rejuvenation
- Muscle Toning
- Eye bags
- Couperose
- Pigmentation
- Hydration
- Lips enhancing

3-day course.



2

Body therapy

- Client consultation
- Body type analysis
- Meridians
- Body zones according to TCM
- Detoxification of the body
- Lymphatic system
- Effect of current on muscles and body
- Slim/Detox
- Cellulite analysis
- Cellulite reduction
- Fat reduction
- Lymphatic drainage
- Sculpting
- Reflexology

3-day course.



4

Hair loss therapy (optional)

- Anatomy of hair
- Hair cycle
- Male hair loss pattern
- Female hair loss pattern
- Alopecia Areata
- Friction-related hair loss
- Scarring hair loss
- Scalp condition: dry, oily, dandruff, seborrheic dermatitis
- Dermaskopi
- Antistress
- Diet
- Case studies
- Client consultation

This is a 2-day special course in hair therapy.

3


Scars and pain therapy

- Acne scars
- Operation scars
- Stretchmarks
- Client consultation
- Pain

*Included in **Beauty Therapy** and **Body Therapy** courses.*

Octagon+ accessories

1. The Octagon+ machine
2. Touch screen stylus
3. Ultratransducer with LED lights
4. Ultrasound Scrubber
5. Reusable and self-adhesive stimulation electrodes
6. Silver Thread Gloves
7. Handheld Dui Rollers in different sizes
8. LED comb (optional)
9. Cryotonic (optional)

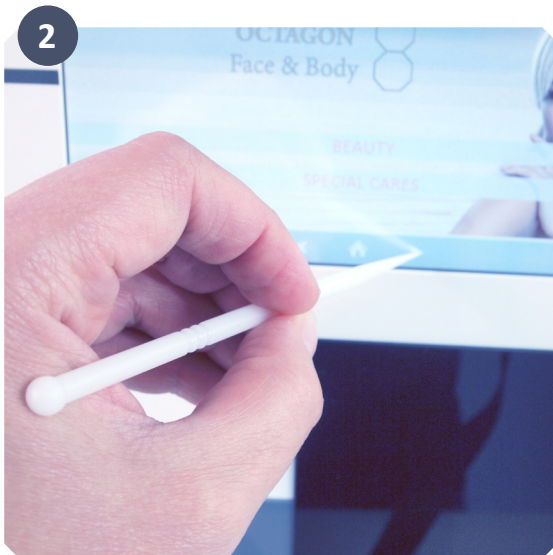

Audrey Christian

OCTAGON+
Face & Body



Dui Clinique AS

Farmannsveien 18-22, 3111 Tønsberg, Norway
Tel: +47 33 33 48 90, Email: post@duiclinique.com
Web: www.duiclinique.no